

APPLICATION FOR OSHPD PREAPPROVAL

OFFICE OF STATEWIDE HEALTH PLANNING AND DEVELOPMENT **FACILITIES DEVELOPMENT DIVISION**

OFFICE USE ONLY

OF MANUFACTURER'S CERTIFICATION (OPM) APPLICATION #: OPM-0339-13										
OSHPD Preapproval of Manufacturer's Certification (OPM)										
ype: ☐ New ☐ Renewal ☐ Update to Pre-CBC 2013 OPA Number:										
Manufacturer Information										
Manufacturer: ADVANCED STERILIZATION PRODUCTS										
Manufacturer's Technical Representative: Curtis Felt										
Mailing Address: 33 Technology Drive, Irvine, CA. 92618										
Telephone: On File Email: On File										
Product Information OS Dod										
Product Name: STERRAD NX Sterilizer with Cart										
Product Type: Other Mechanical and Electrical Components										
Product Model Number: STERRAD NX BY: William Staehlin										
General Description: Compact sterilizer for medical instruments										
DATE: 09/11/2017										
Applicant Information										
Applicant Company Name: EASE Co.										
Contact Person:Jonathan Roberson, S.E.										
Mailing Address: 5877 Pine Ave. Suite 210, Chino Hills, CA. 91709										
Telephone: _(909) 606-7622										
I hereby agree to reimburse the Office of Statewide Health Planning and Development review fees in accordance with the California Administrative Code, 2016.										
Signature of Applicant: Date: 04/18/16										
Title: Principal Engineer Company Name: EASE Co.										

"Access to Safe, Quality Healthcare Environments that Meet California's Diverse and Dynamic Needs"





Page 1 of 2



OFFICE OF STATEWIDE HEALTH PLANNING AND DEVELOPMENT **FACILITIES DEVELOPMENT DIVISION**

Registered Design Professional Preparing Engineering Recommendations									
Company Name: EASE Co.									
Name: Jonathan Roberson, S.E. California License Number: S4197									
Mailing Address: 5877 Pine Ave. Suite 210, Chino Hills, CA. 91709									
Telephone: 909-606-7622 Email: <u>J.Roberson@EASECo.com</u>									
OSHPD Special Seismic Certification Preapproval (OSP)									
 □ Special Seismic Certification is preapproved under OSP-(Separate application for OSP is required) □ Special Seismic Certification is not preapproved 									
Certification Method(s)									
☐ Testing in accordance with: ☐ ICC-ES AC156 ☐ FM 1950-16 ☐ Other* (Please Specify):									
OPM-0339-13									
*Use of criteria other than those adopted by the California Building Standards Code, 2016 (CBSC 2016) for component supports and attachments are not permitted. For distribution system, interior partition wall, and suspended ceiling seismic bracings, test criteria other than those adopted in the CBSC 2016 may be used when approved by OSHPD prior to testing. Analysis DATE: 09/11/2017									
Experience Data									
Combination of Testing, Analysis, and/or Experience Data (Please Specify):									
COD'									
List of Attachments Supporting the Manufacturer's Certification									
☐ Test Report☐ Other(s) (Please Specify):☐ Other(s) (Please Specify):									
OFFICE USE ONLY – OSHPD APPROVAL VALID FOR CBC 2016 & ALL PRE-2016 CODE BASED PROJECTS									
Signature: Date: 09-11-2017									
Print Name: William Staehlin									
Title: SSE									
Condition of Approval (if applicable):									

"Access to Safe, Quality Healthcare Environments that Meet California's Diverse and Dynamic Needs"







EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

5877 Pine Ave, Ste. 210 Chino Hills, CA. 91709 Phn: (909) 606-7622

Office of Statewide Health Planning and Development

PREAPPROVAL OF MANUFACTURER'S CERTIFICATION OPM-0339-13

THIS PREAPPROVAL CONFORMS TO THE 2016 CALIFORNIA BUILDING CODE

MANUFACTURER: EQUIPMENT NAME:

ADVANCED STERILIZATION PRODUCTS

STERRAD NX WITH ALLCLEAR STERILIZER WITH CART

Sheet: 1 of 13 Date: 9/7/17

GENERAL NOTES

- 1. THIS OSHPD PREAPPROVAL OF MANUFACTURER'S CERTIFICATION (OPM) IS BASED ON THE 2016 CBC. THE DEMANDS (DESIGN FORCES) FOR USE WITH THIS OPM SHALL BE BASED ON THE 2016 CBC
- 2. THIS DOCUMENT MAY ONLY BE USED WITH THE EXPRESS WRITTEN CONSENT OF THE MANUFACTURER LISTED ABOVE FOR THE SPECIFIC PROJECT SITE AND INSTALLATION LOCATION. THIS DOCUMENT IS INVALID WITHOUT SUCH CONSENT.
- 3. THIS PREAPPROVAL CONFORMS TO THE 2016 CALIFORNIA BUILDING CODE WHERE SDS IS NOT GREATER THAN 2.20. SEE DETAIL FOR APPLICABILITY OPM-0339-13
- 4. FORCES PER ASCE 7-10 SECTION 13:3:1, EQUATIONS 13:3-1, 13:3-2 & 13:3-3,

 WHERE SDS = 2.20, a_p = 1.0, I_p = 1.5, R_p = 1.5, z/h_j 0 AT CONCRETE SLAB, z/h ≤ 1 AT CONCRETE SLAB ON METAL DECK.

 SEE FOLLOWING SHEETS FOR Ω₀
- 5. THIS PREAPPROVAL COVERS ONLY THE SUPPORTS AND ATTACHMENTS OF THE EQUIPMENT TO THE STRUCTURE.
- ALL DESIGN FORCES SHOWN ON THE DRAWINGS ARE FACTORED LOADS THAT SHALL BE USED FOR STRENGTH DESIGN.
- 7. CONCRETE SLAB ON METAL DECK DETAIL VALID FOR DEMANDS SHOWN AT ANY ELEVATION IN THE BUILDING. (i.e. z/h < 1)
- 8. CONCRETE SLAB DETAIL VALID FOR DEMANDS SHOWN AT ANY ELEVATION AT OR BELOW GRADE. (i.e. z/h = 0)

9. RESPONSIBILITIES OF THE STRUCTURAL ENGINEER OF RECORD OF THE BUILDING

- A. PROVIDE SUPPORTING STRUCTURE TO SUPPORT WEIGHTS AND FORCES SHOWN IN ADDITION TO ALL OTHER LOADS.
- B. VERIFY THAT THE INSTALLATION IS IN CONFORMANCE WITH THE 2016 CBC AND WITH THE DETAILS, MATERIAL AND GAGE OF THE UNIT WHERE ATTACHMENTS ARE MADE AGREE WITH THE INFORMATION SHOWN ON THE PREAPPROVAL DOCUMENTS.
- C. VERIFY THAT PROJECT SPECIFIC VALUES OF SDS & z/h RESULT IN SEISMIC FORCES (Eh, Ev) THAT DO NOT EXCEED THE VALUES ON THE DETAILS.
- D. VERIFY THAT THE CONCRETE SLAB TO WHICH THE EQUIPMENT IS ANCHORED MEETS THE REQUIREMENTS OF THE APPLICABLE ICC ESR AND THIS OPM.
- E. VERIFY THAT THE ANCHORS ARE AN ADEQUATE DISTANCE FROM ANY SLAB EDGES OR OPENINGS (SEE TYPICAL DETAIL ON SHEET 2).
- F. VERIFY THAT ALL NEW OR EXISTING ANCHORS ARE AN ADEQUATE DISTANCE FROM THE UNIT ATTACHMENTS AND CHECK FOR INTERACTION WHERE OTHER ANCHORS ARE WITHIN 18" OR 6hef FROM THIS UNIT'S ANCHORS.



EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

www.EquipmentAnchorage.com

ADVANCED STERILIZATION PRODUCTS

STÉRRAD NX WITH ALLCLEAR STÉRILIZER WITH CART

DES. J. ROBERSON

JOB NO. 11-1517

DATE 9/7/17

SHEET 2

F 13 SHEETS

10. EXPANSION ANCHORS:


A. ATTACHMENT IS TO BE MADE WITH THE ANCHORS LISTED BELOW AND INSTALLED AS DESCRIBED IN THE CORRESPONDING ICC REPORT.

Anchor Diameter	Concrete Type	Min. f'c (psi)	Anchor Type	ICC Report No.	Min. Embed.	Min. Spacing	Min. Edge Dist.	Min. Conc. Thickness	Torque Test	Direct Tension
3/8"	Sand Light Weight	3000	Hilti Kwik Bolt TZ	ESR-1917	2"	6.75"	12"	See Detail "A"	25 FT-LB	N / A
3/8"	Normal Weight	3000	Hilti Kwik Bolt TZ	ESR-1917	2"	3"	12"	4"	40 FT-LB	1605 lb

- B. THIS PREAPPROVAL ALLOWS FOR UP TO A MAXIMUM OF 2 ADJACENT CONCRETE SLAB EDGES, 12" AWAY MINIMUM (i.e. CORNER).

 SEE ADJACENT DETAIL FOR ADDITIONAL MINIMUM ALLOWABLE CONCRETE EDGE DISTANCES.
- C. TESTING OF EXPANSION ANCHORS PER 2016 CBC, 1910A.5:

 TESTING SHALL BE DONE IN THE PRESENCE OF THE SPECIAL INSPECTOR AND A REPORT OF THE TEST RESULTS SHALL BE SUBMITTED TO OSHPD
 - (i) AFTER AT LEAST 24 HOURS HAVE ELAPSED SINCE INSTALLATION,
 DIRECT PULL TENSION TEST OR TORQUE TEST AT LEAST 50% OF
 THE ANCHORS.
 - (ii) ACCEPTANCE CRITERIA:
 - DIRECT TENSION TEST: THE ANCHOR SHOULD HAVE NO OBSERVABLE MOVEMENT AT THE TEST LOAD. A PRACTICAL WAY TO DETERMINE OBSERVABLE MOVEMENT IS THAT THE WASHER BECOMES LOOSE.
 - TORQUE TEST: THE APPLICABLE TORQUE MUST BE ACHIEVED
 WITHIN THE FOLLOWING LIMITS: WEDGE TYPE: 1/2 TURN OF THE
 NUT
 - (iii) IF ANY ANCHOR FAILS, TEST ALL ANCHORS.
- 11. BOLTS THROUGH CONCRETE ON METAL DECK
 - A. BOLTS SHALL BE TORQUED BY 3/4 TURN OF THE NUTS AFTER THE SNUG TIGHT (THE SNUG-TIGHT CONDITION IS DEFINED AS THE TIGHTNESS REQUIRED TO BRING THE CONNECTED PLIES INTO FIRM CONTACT) CONDITION IS ACHIEVED. UNLESS OTHERWISE NOTED.
 - B. THROUGH BOLT HOLES SHALL BE 1/16" LARGER THAN BOLT SIZE (HOLE SIZE = BOLT SIZE + 1/16) FOR CONCRETE.
 - C. THROUGH-BOLTS IN CONCRETE SHALL RECEIVE SPECIAL INSPECTION AND TESTING (THROUGH BOLTS WITH STEEL TO STEEL CONNECTION IN TENSION DO NOT REQUIRE TENSION TESTING) IN ACCORDANCE WITH REQUIREMENTS FOR POST-INSTALLED ANCHORS.

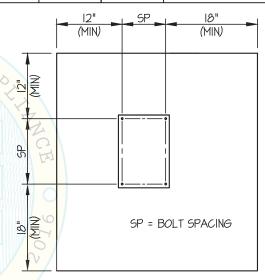

TYPICAL CONCRETE EDGE DETAIL



EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

www.EquipmentAnchorage.com

ADVANCED STERILIZATION PRODUCTS

STÉRRAD NX WITH ALLCLEAR STERILIZER WITH CART

DES. J. ROBERSON

ЈОВ NO. 11-1517


DATE 9/7/17

SHEET 3

of 13 SHEETS

SEISMIC SUPPORTS & ATTACHMENTS

CONCRETE SLAB

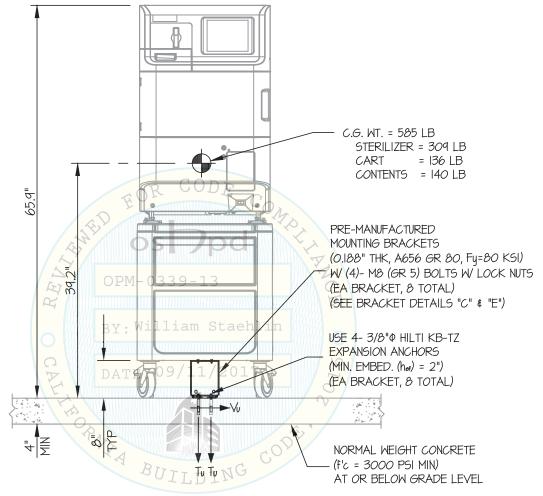

Tu = 761 LB/BOLT (MAX) Vu = 147 LB/BOLT (MAX)(VALUES INCLUDE Ω_0)

FRONT ELEVATION

NOTES:

1. FORCES ARE DETERMINED PER 2016 CALIFORNIA BUILDING CODE AND ASCE 7-10

STRENGTH DESIGN IS USED. (SDS = 2.20, Δp = 1.0, lp = 1.5, Rp = 1.5, Ω_0 = 1.5, z/h = 0)

HORIZONTAL FORCE (En) = 0.99 Wp

HORIZONTAL FORCE (Emh) = 1.49 Wp (FOR CONCRETE ANCHORAGE)

VERTICAL FORCE (Ev) = 0.44 Wp

2. CENTER OF GRAVITY (C.G.) AND WEIGHT ARE THE GOVERNING PARAMETERS FOR DESIGN. THESE CALCULATIONS ENCOMPASS ALL WEIGHTS UP TO THE MAXIMUM WEIGHT SHOWN.

3. STRUCTURAL ENGINEER OF RECORD FOR THE BUILDING SHALL PROVIDE SUPPORT STRUCTURE DESIGNED TO SUPPORT WEIGHTS AND FORCES SHOWN IN COMBINATION WITH ALL OTHER LOADS THAT MAY BE PRESENT.

4. SEE GENERAL NOTES: SHEETS 1 AND 2

EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

www.EquipmentAnchorage.com

ADVANCED STERILIZATION PRODUCTS


STÉRRAD NX WITH ALLCLEAR STÉRILIZER WITH CART


DES. J. ROBERSON

JOB NO. 11-1517

DATE 9/7/17

SHEET

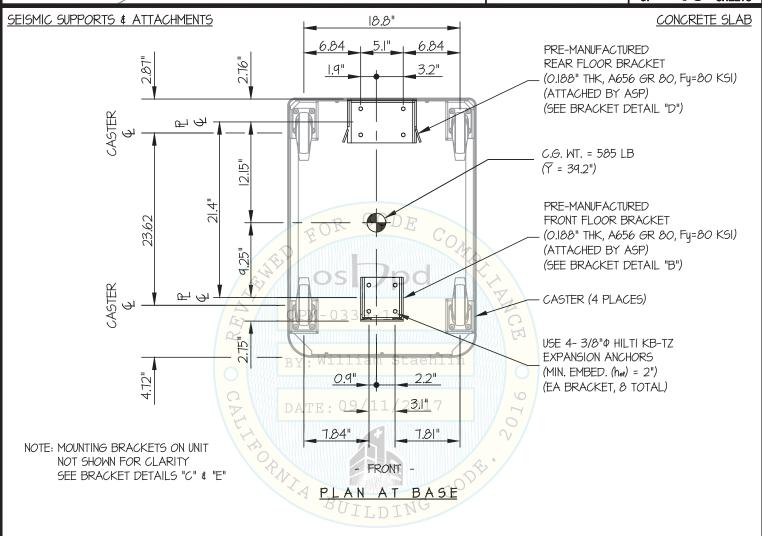

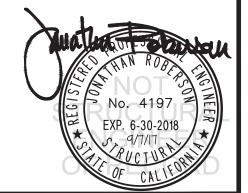

EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

www.EquipmentAnchorage.com

ADVÁNCED STERILIZATION PRODUCTS

STÉRRAD NX WITH ALLCLEAR STERILIZER WITH CART

DES. J. ROBERSON

11-1517 JOB NO.


9/7/17 DATE

SHEET

SHEETS

SEISMIC SUPPORTS & ATTACHMENTS

CONCRETE SLAB ON METAL DECK

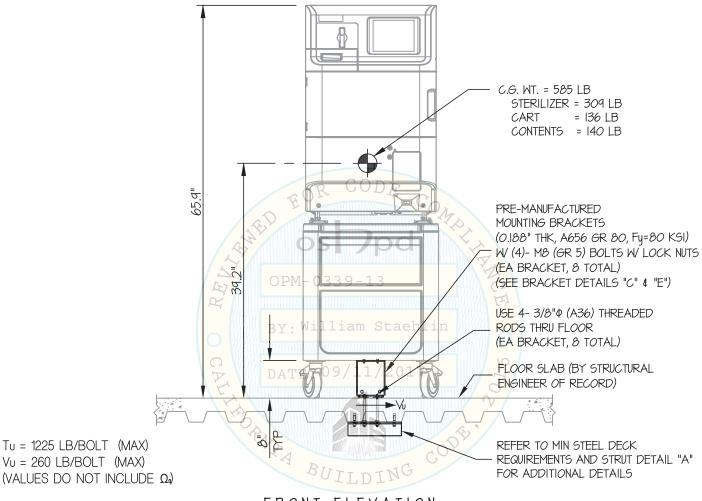

FRONT ELEVATION

NOTES:

Vu = 260 LB/BOLT (MAX)

FORCES ARE DETERMINED PER 2016 CALIFORNIA BUILDING CODE AND ASCE 7-10.

STRENGTH DESIGN IS USED. (SDS = 2.20, Δp = 1.0, lp = 1.5, Rp = 1.5, Ω_0 = 1.5, z/h < 1)

HORIZONTAL FORCE (En) = 2.64 Wp

HORIZONTAL FORCE (Emh) = 3.96Wp (FOR CONCRETE ANCHORAGE)

VERTICAL FORCE (E_V) = 0.44 W_p

2. CENTER OF GRAVITY (C.G.) AND WEIGHT ARE THE GOVERNING PARAMETERS FOR DESIGN, THIS PREAPPROVAL ENCOMPASSES ALL WEIGHTS UP TO THE MAXIMUM WEIGHT SHOWN.

3. STRUCTURAL ENGINEER OF RECORD FOR THE BUILDING SHALL PROVIDE SUPPORT STRUCTURE DESIGNED TO SUPPORT WEIGHTS AND FORCES SHOWN IN COMBINATION WITH ALL OTHER LOADS THAT MAY BE PRESENT.

4. SEE GENERAL NOTES: SHEETS 1 AND 2



EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

www.EquipmentAnchorage.com

ADVANCED STERILIZATION PRODUCTS

STÉRRAD NX WITH ALLCLEAR STÉRILIZER WITH CART


DES. J. ROBERSON

JOB NO. 11-1517

DATE 9/7/17

SHEET

6

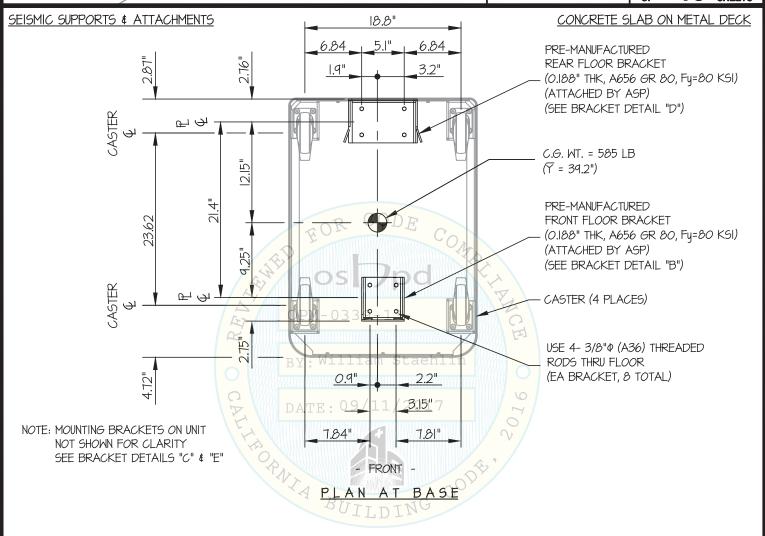



EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

www.EquipmentAnchorage.com

ADVANCED STERILIZATION PRODUCTS

STÉRRAD NX WITH ALLCLEAR STÉRILIZER WITH CART


DES. J. ROBERSON


JOB NO. 11-1517

DATE 9/7/17

SHEET 7

13 SHEETS

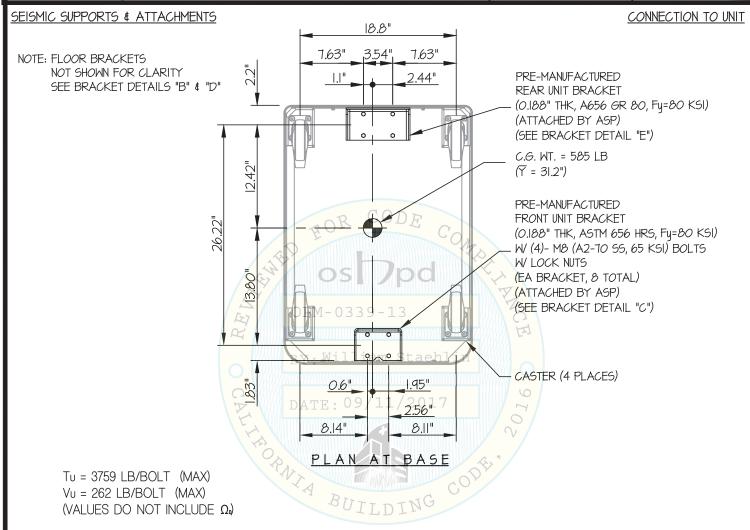

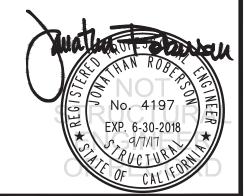

EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

www.EquipmentAnchorage.com

ADVANCED STERILIZATION PRODUCTS

STÉRRAD NX WITH ALLCLEAR STERILIZER WITH CART

DES. J. ROBERSON

JOB NO. 11-1517


DATE 9/7/17

8HEET

13 SHEETS

SEISMIC SUPPORTS & ATTACHMENTS

CONCRETE SLAB ON METAL DECK

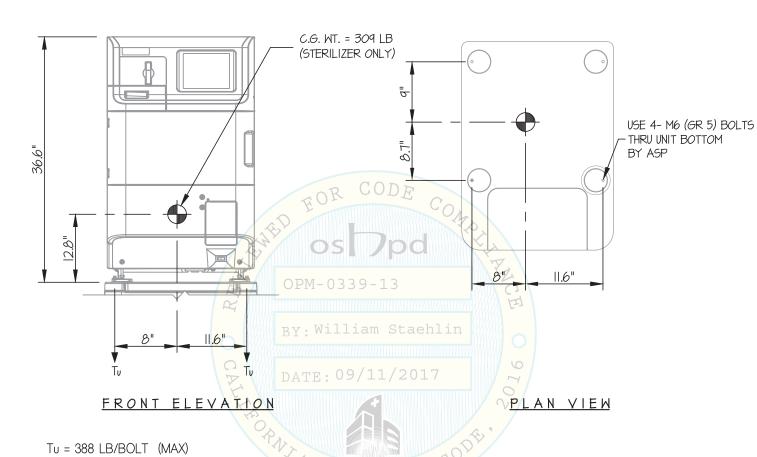

No. 4197

EXP. 6-30-2018

**C - 9/7/17

PUCTUSE

OF CALLS

Vu = 304 LB/BOLT (MAX)

(VALUES DO NOT INCLUDE Ω)

BUILDING

EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

www.EquipmentAnchorage.com

VANCED STERILIZATION PRODUCTS

STÉRRAD NX WITH ALLCLEAR STERILIZER WITH CART

DES. J. ROBERSON

11-1517 JOB NO.

9/7/17 DATE

SHEET

SHEETS OF

SEISMIC SUPPORTS & ATTACHMENTS

CONCRETE DETAILS 32" MIN EDGE DISTANCE 3" ONCRETE EDGE Θ Θ N.W. OR SAND L.W. MIN FLUTE CONC. (3000 PSI MIN.) 12" (MIN) EQUIPMENT-TYP 3 · 4 · · · · 4 I" MAX **OFFSET** Vu STRUT MIN 20 GA (TYP) STEEL W-DECK 45" I" MIN FLUTE USE 3/8" PHILTI KB-TZ TYP EXPANSION ANCHORS Ł (MIN, EMBED, (hef) = 2") HEX NUT TOP & BOT OF FLANGE FLUTE DETAIL (2 ANCHORS MIN PER STRUT) (TYP) AT CONDITIONS WHERE NUT CANNOT BE PROVIDED AT TOP SIDE

OF STRUT, PROVIDE TAPPED HOLE

THROUGH STRUT FLANGE.

(A36) AT EACH ANCHOR (EXTEND ANGLE TO ADJACENT FLUTE WHEN THREADED ROD OCCURS AT FLUTE)

L3 X 3 X I/4" X I'-2" MIN

MIN STEEL DECK REQUIREMENTS AND STRUT DETAIL

ZA BUILDING



EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

www.EquipmentAnchorage.com

ADVANCED STERILIZATION PRODUCTS


STÉRRAD NX WITH ALLCLEAR STERILIZER WITH CART


DES. J. ROBERSON

JOB NO. 11-1517

DATE 9/7/17

SHEET 1

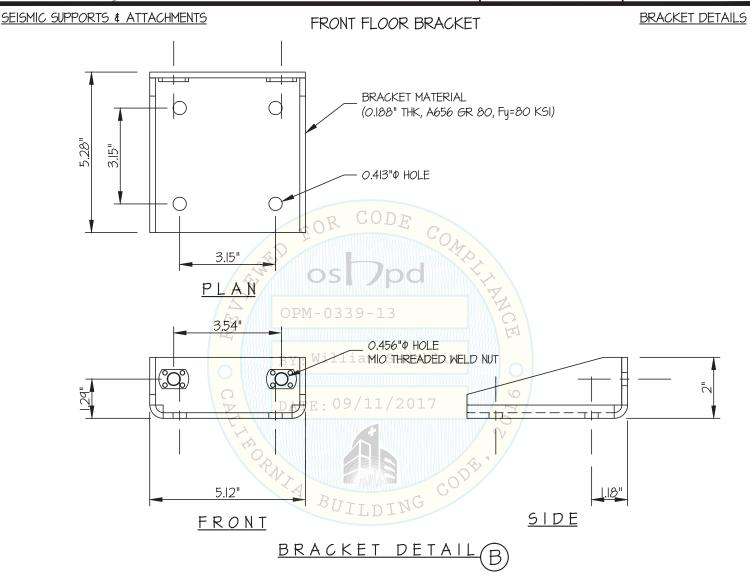

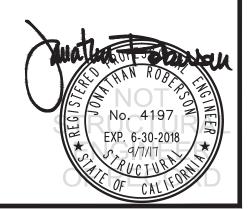

EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

www.EquipmentAnchorage.com

ADVANCED STERILIZATION PRODUCTS


STÉRRAD NX WITH ALLCLEAR STERILIZER WITH CART

DES. J. ROBERSON

JOB NO. 11-1517

DATE 9/7/17

SHEET 1 1

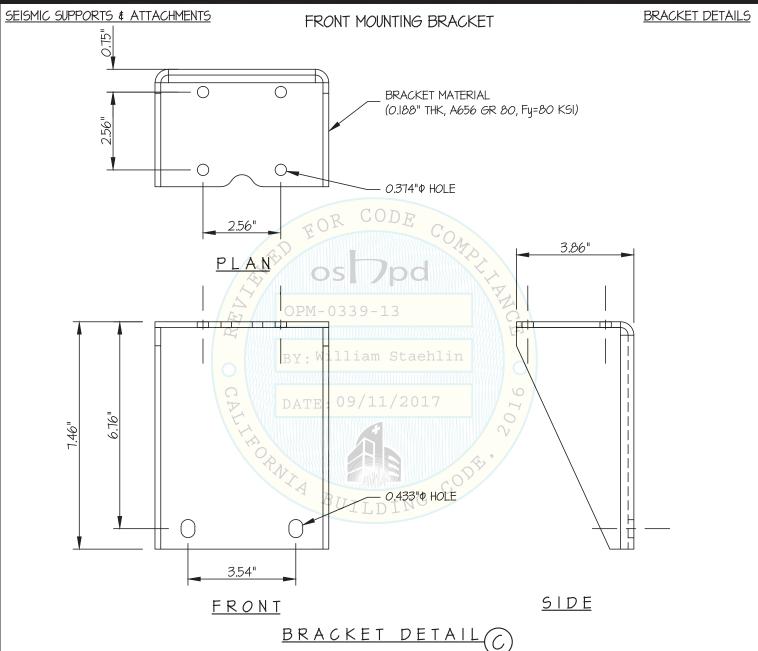

EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

www.EquipmentAnchorage.com

ADVANCED STERILIZATION PRODUCTS


STÉRRAD NX WITH ALLCLEAR STERILIZER WITH CART

DES. J. ROBERSON

JOB NO. 11-1517

DATE 9/7/17

1 **2**

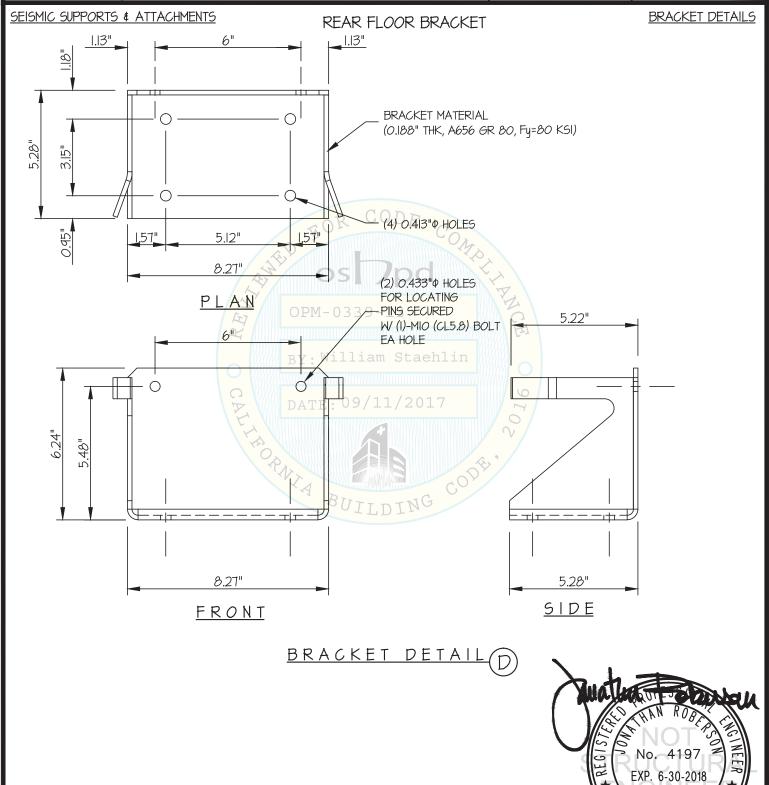

www.EquipmentAnchorage.com

ADVANCED STERILIZATION PRODUCTS

STÉRRAD NX WITH ALLCLEAR STÉRILIZER WITH CART

DES. J. ROBERSON

JOB NO. 11-1517

DATE 9/7/17

13

of 13 SHEETS

SEISMIC SUPPORTS & ATTACHMENTS

REAR MOUNTING BRACKET

BRACKET DETAILS

